## **PREVNER®** Koortslipcrème

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READ THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE USING PREVNER. KEEP THIS LEAFLET FOR REFERENCE; IT MAY BE NECESSARY TO READ THE INFORMATION AGAIN.

### **What is Prevner®?**

Prevner® is a lip cream for the prevention of cold sores. It protects the lips against development of a sore by blocking out ultraviolet light. Additionally, it nurtures the skin, preventing the lips from drying out. Prevner® is a medical device intended for adults and children ages 4 years and older.

### **When to use Prevner®:**

Apply Prevner® preventively whenever you are at an increased risk of developing a cold sore. These may be situations of high physical strain or mental stress, for instance while hiking or skiing, before tense deadlines or on business trips, during winter or the flu season, and so on. Most persons suffering from cold sores recognize an oncoming outbreak by a tingling or swelling sensation in the lips, which then is the critical moment for applying Prevner® and prevent a sore on the mouth.

### **How to use Prevner®:**

Apply Prevner® 4 - 6 times per day, using a cleaned finger. A dosage the size of half a pea may be sufficient to protect a single affected area. **Tip:** to ensure protection throughout the day, it may help to set regular time of application, such as after each meal, after a bath, when leaving a building, etc., or instead you may choose to set an alarm at a certain time interval.

### **What causes cold sores?**

Cold sores are caused by herpes simplex virus (type 1 or 2), silently hidden in nerves connecting to the mouth. The virus is equipped to be triggered into activity by certain states of our natural defense systems following stress and tissue damage, and to exploit these systems by rapidly reproducing near the surface of the mouth. Any resulting open lesions of the skin facilitate the spread of the virus to another host. The remaining lip sore then takes 1 - 2 weeks to scab over and heal.

### **How does Prevner® work?**

Prevner® works by blocking the stimuli for viral reactivation. It filters out damaging UV-light using titanium dioxide, aided by bemotrizinol for broad-spectrum UV-protection. Moreover, zinc sulfate and allantoin condition the skin to further protect the lips from harm. Rather than using pharmacological substances, Prevner® prevents the development of a cold sore by physical means.

**Do not use Prevner®** if you are allergic or hypersensitive to any of its ingredients.

### **Warnings:**

- For personal use only; to prevent spread of the virus, do not share among other persons.
- For external use - on the lips - only.
- Keep out of reach and sight of children.
- Always close the tube after use.
- Store at 5 - 25°C.
- Do not use beyond the expiry date (see tube or box).
- Warning! Hazardous respirable droplets may be formed when sprayed. Do not breathe spray or mist.

**Notice:** please be sure to notify both the manufacturer and your national health authority on medical devices in case of any serious incidents related to use of Prevner®.

**Ingredients:** water, isopropyl myristate, white petrolatum, propylene glycol, titanium dioxide and silica (nano), cetearth-20, cetearyl alcohol, paraffin oil, bemotrizinol, phenoxyethanol, zinc sulfate, tocopheryl acetate, mentha piperita oil, allantoin and hinokitiol.

**Fabrikant/Manufacturer:**  
Pronova Laboratories BV  
Kruitpad 16  
1398 CP Muiden  
Nederland/The Netherlands

PN15NL100  
Last revised: 05/2023